



Assured Equipment Servicing And Repairs



JustWorks


Equipment Service Plan

In your busy vet practice, equipment downtime can impact directly on patient care and outcomes.

What is JustWorks?

JustWorks from Intriquip is a service designed to help veterinary professionals avoid unnecessary downtime through timely, expert servicing, maintenance and repairs on a wide variety of veterinary equipment.

We're factory-approved service agents for many of the veterinary equipment brands we supply.

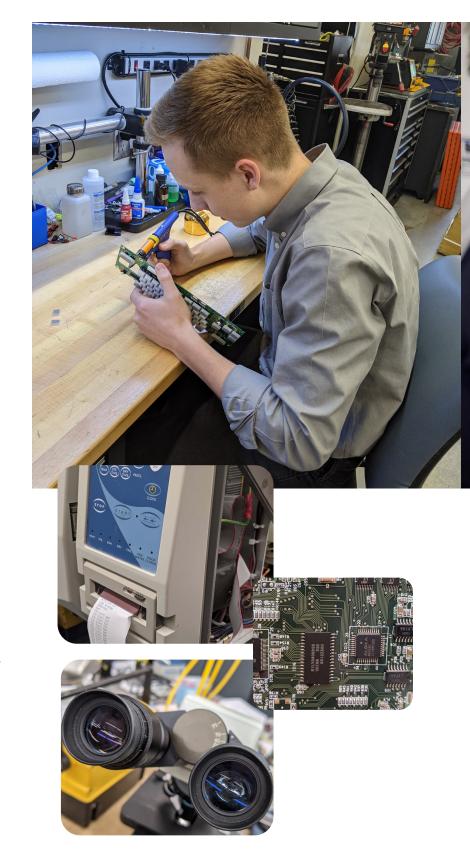

JustWorks offers a range of flexible service plans designed to protect your practice from unexpected costs and disruption.


We offer servicing and emergency repairs for

- Anesthesia systems
- Waste gas evacuation systems
- Dental delivery systems
- Steam Sterilizers (autoclaves)
- Microscope
- Oxygen Concentrators
- X-ray (Whole Body) systems
- CR and DR Digital X-ray Acquisition systems
- Dental X-ray generators
- Centrifuges
- Ultrasound Scanners



1.800.361.3777 service@intriquip.com

intriquip.com

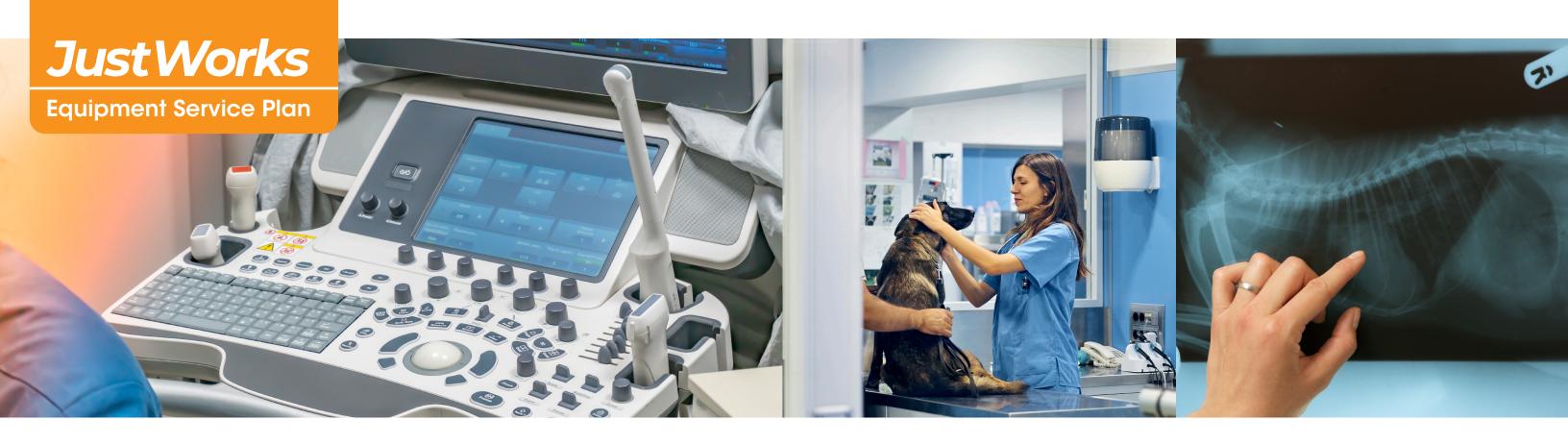

JustWorks for you

Flexible Plans To Suit Your Needs

Simply choose the right plan for your practice based on your equipment and required servicing frequency

Specialist, Manufacturer Endorsed Service

All JustWorks servicing and repairs are carried out by experienced technicians. We're an approved service partner for many of the equipment brands we supply, enabling us to carry out warranty repairs.satisfaction guarantee.

We also perform warranty repairs, preventative maintenance services and X-ray compliance inspections.





Eradicate Downtime

Flexible Plans To Suit Your Needs

Our free loaner service means that if we can't repair your faulty machinery there and then, we'll lend you an equivalent replacement so that your work can continue uninterrupted.

Logners available include

- Anesthesia
- Autoclaves
- Dental Delivery Systems
- Microscopes
- Portable X-Ray
- Ultrasounds

Value For Money

We offer free repair estimates and discounted labour rates to help you get the best deal for your business. We also offer free phone support and troubleshooting to help you get answers, fast.













Complete Maintenance **Experts**

Regular servicing keeps veterinary equipment in tip-top condition, for first class care and total peace of mind.

Anesthesia System

- Multi point system inspection
- Vaporizer calibration check (required every 2 years, recommended annually)
- Svstem leak tests
- Cleaning

Waste Gas Evacuation Systems

 Verify positive gas evacuation flow at each anesthesia delivery location

Dental Delivery System

- Compressor Oil Change
- Replace inlet air filter if applicable
- Pressure settings checked and adjusted
- Safety pressure valve check
- Drain moisture from air pressure tank
- Hand pieces, function tested, water flow verified where applicable

Steam Sterilizers

- Replace door aasket
- Documented multi point condition inspection
- Cleaning cycle
- Test run with verification of temperature and pressure

Microscopes

- Replace light bulb (if halogen)
- Clean Exterior and Optics
- Lubricate bearings and slides as needed
- Reset Kohler Alignment if applicable
- Test to confirm proper functionality

Oxvaen Concentrator

- Supera units (we supply fliters)
- Other brands (customer to order filters elsewhere)
- Install new air filters
- Test output supply for purity and volume

Centrifuges

- Clean and check condition of lid lock, rotor and tube carriers
- Certify RPM and timer



- Whole Body CR
- Test scans for normal function
- Roller cleaning and cooling fan cleaning
- Reassemble
- Test Images of certified x-ray test images for consistency,



- Collimator bulb replaced (if Halogen)
- Collimator alignment checked
- (high tension lead connectors)
- Test kVp, Timer and mA outputs
- (5th year) Complete SPM Inspection and Document Submittal (SK and MB) or Inspection Readiness Test (AB)



- Cover off internal inspection
- phantom (comparison to previous noise, artifacts etc..)
- Computer display light output verified with inspection light meter





- Inspection for damaged covers, external damage, positioning arm wear/tear
- Check positioning arm for proper positioning/drift control. Adjust as needed
- Check control panel for damage and normal functionality
- Test outputs (kVp, mA and timer)
- (5th year) Complete SPM Inspection and Document Submittal (SK and MB) or Inspection Readiness Test (AB)

Ultrasounds

- Clean computer cooling system
- Check electrical safety
- Check battery condition
- Check image quality and measurement function using phantom

Whole Body DR

- Test Images of certified x-ray phantom (comparison to previous test images for consistency, noise, artifacts etc..)
- Computer display light output verified with inspection light meter





1.800.361.3777

service@intriquip.com

For All Your Veterinary Equipment And Servicing Needs

Contact Us Today
To Discuss Your Needs.



1.800.361.3777

service@intriquip.com

intriquip.com